



TECHNICAL UPDATE

Cross-reactivity Information for Immunoassay Screening Tests

TEST	POSITIVE DRUGS	FALSE POSITIVES
Amphetamines	Amphetamine Methamphetamine MDMA (Ecstasy) MDA (Sass, Pink, Yop) MDEA (EVE) MBDB (Eden) BDB, PMMA, PMA	
Barbiturates	Amobarbital Butalbital Butabarbital Pentobarbital Phenobarbital Secobarbital	
Benzodiazepines	Alprazolam Bromazepam Chlordiazepoxide Clobazam Clonazepam Clorazepate Delorazepam	Oxaprozin (Daypro®)

TEST	POSITIVE DRUGS	FALSE POSITIVES
Benzodiazepines (Continued)	Diazepam Flunitrazepam Flurazepam Lorazepam Lormetazolam Nitrazepam Nordiazepam Norfludiazepam Oxazepam Prazepam Temazepam Triazolam	
Buprenorphine	Buprenorphine (Suboxone®)	Trimethoprim and Sulfamethoxazole (Bactrim®)
Cannabinoids	9-Carboxy-THC (Marijuana Metabolite)	
Cocaine Metabolite	Benzoylcegonine (Cocaine Metabolite)	
Ethyl Alcohol	Ethanol (Alcohol)	
Methadone	Methadone	Doxylamine
Methadone Metabolite	EDDP (Methadone Metabolite)	
Opiates	Morphine Codeine Dihydrocodeine Hydrocodone (Vicodin®) Hydromorphone (Dilaudid®) Oxycodone (Elevated Concentrations) Oxymorphone (Elevated Concentrations)	

Oxycodone	Oxycodone (Oxycontin®, Percodan®) Oxymorphone	
Phencyclidine (PCP)	PCP (Angel Dust)	Doxylamine
TEST	POSITIVE DRUGS	FALSE POSITIVES
Propoxyphene	Propoxyphene (Darvon®) Norpropoxyphene (Darvon® Metabolite)	
Tricyclic Antidepressants	Amitriptyline Clomipramine Desipramine Doxepin Imipramine Nordoxepin Nortriptyline Protriptyline Trimipramine	Chlorpromazine Cyclobenzaprine

Please understand that the drug testing field is constantly undergoing changes to keep up with new developments. If a drug is not listed here, feel free to contact one of our laboratory professionals for additional information. 866-857-8378